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## IMPORTANT SAFETY INSTRUCTIONS

1. **Read the instruction** – All the safety and operating instructions should be read before using.
2. **Keep the instruction** – The instruction manual should be kept as reference for safety purposes.
3. **Follow all instructions** – All operating and use instructions should be followed.
4. **Clean only with clean cloth and use products suggested by the dental industry.**
5. **Only use attachments/accessories specified by the manufacturer.**
6. **For service, refer to qualified personnel.**

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## ÉTAPE 2 / STEP 2

### TABOURET ASSISTANTE / ASSISTANT STOOL

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## ADJUSTMENT

- Ø The height adjustment of stool was made by an adjustment handle under the seat in right side.
- Ø The height adjustment of abdominal support arm of assistant stool was made by a handle on the arm.
- Ø The opening and closing of the abdominal support arm were made by a ratchet handle under the cushion near the arm.
- Ø The height adjustment of the back rest was made with the handle on the back of this one.
- Ø The depth adjustment of the backrest is using the tightening handle under the seat

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## DISINFECTION GUIDELINES

Safari Dental makes no representation as to the disinfection efficacy of any disinfectant product. Questions regarding product efficacy should be directed to the specific manufacturer of that product. We make no warranty expressed or implied regarding any damage resulting from the use of any disinfectant. Damage and/or discoloration of any surface is not covered by our warranty.

There are many products and techniques for infection control that are presently available on the market.

The **BARRIER TECHNIQUE METHOD** has proved to be effective in the preservation of the surface finish and appearance of your new equipment. Disposable barrier products should be used and changed between each patient. Consult your full-service dental dealer for information regarding disposable barrier products.

The **CHEMICAL DISINFECTION METHOD** is an effective means of disinfection, but care is needed in the mixing and use of the products. To minimize the accumulation of chemical residues on your new equipment, you must wash your equipment daily with a solution of warm water and mild soap. When mixing any cleaning or disinfecting chemical product, care should be taken to follow the manufacturer's instructions regarding mixing and use. If recommended ratios of dilution are not followed, non-damaging chemical may become corrosive and damage surface finishes. Consult your full-service dental dealer for information regarding chemical disinfecting and cleaning products.

Non-Acceptable disinfectants and cleaners include: Household bleach @ 5.25%, Ethyl alcohol, alcohol, alcohol-based phenols, household cleaners and strong phenols or alcohol-phenol combinations. The use of iodine-based products has been known to stain painted and upholstered surfaces.

**DAMAGE OR DISCOLORATION OF SURFACE FINISHES IS NOT COVERED UNDER THE SAFARI DENTAL WARRANTY.**

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## REPAIR RETURN PROCEDURE

Toll free: 1-800-567-0013 or 450-433-5754  
Email: sales@safaridental.com

We will assist you or direct you to return the product with a copy of the original purchase invoice to the nearest service center.

Pack your product carefully to avoid damage in shipping. Only send the defective part. Keep all removable parts to avoid further damage during the transport.

If you need to return your unit, please include the following instructions:

- Ø A brief description of the problem
- Ø A contact name
- Ø A contact telephone number, fax number or email address
  
- Ø A complete address for the return of the unit
- Ø If returned under warranty repair, please provide proof of claim

Upon receipt of the unit, our technicians will inspect the unit and you will be provided with an estimate of repair. The purchaser shall bear all shipping, packing and insurance costs. All repair work to be done after the warranty period, our service center will bill you for material and labor only on a flat rate repair fee basis.

If you have any question, please feel free to contact us. Toll free: 1-800-567-0013 or 450-433-5754  
Email: sales@safaridental.com