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IMPORTANT SAFETY INSTRUCTIONS

1. **Read these instructions** – All these safety and operating instructions should be read before the unit is operated.
2. **Keep these instructions** – The safety, operating and use instructions should be retained for reference.
3. **Heed all warnings** – All warning on the product and in the operating instructions should be adhered to.
4. **Follow all instructions** – Instructions should be followed.
5. **Clean only with a clean cloth and products authorized by the manufacturer** – Unplug this unit from the wall outlet before cleaning. Only use cleaning products used in dental and medical offices.
6. **Between each patient, use maximum 2 ounces of NON-FOAMING PRODUCT (half glass DEXIE-CUP) per suction tubing, and the same quantity of water. Finally empty the waste reservoir.**
7. **Do not exceed 2/3 of the waste reservoir capacity**
8. **Handle this unit with care** – Do not hit or damage this unit during transportation.
9. **Do not install near any heat sources such radiators, heat registers, stoves, or other apparatus that produce heat.**
10. **Only use attachments/accesories specified by the manufacturer.**
11. **Refer all servicing to qualified service personnel.**

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INSTALLATION AND INITIAL REGULATINGS

Note :It is better to retain the delivery box, since they could be useful if you had to return the unit for service .

1. **Put the unit on a stable surface and open the door** – Remove with care the air/water syringe tip and the (SE) valve. Remove also the foot speed control (reostat).
2. **Water bottle preparation** - Remove the bottle for cleaning (see cleaning instructions page 5). Fill the bottle with clean water or preferably demineralized water. Replace the bottle and tighten gently.
3. **Waste reservoir preparation** – Unscrew the reservoir. Clean the waste tank (See instructions for cleaning). Put reservoir back on with your hands . **Put the suction tube back. Replace the suction valve on the 4 positions instrument support.**

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INSTRUCTIONS UTILISATION

Syringe air/water

1. Plug the unit
2. Check jet of air, water, and air / water spray when the compressor is activated.

Oral evacuation system

1. Plug the unit
2. To activate and deactivate, place the suction control switch.
3. Never fill the waste reservoir at its full capacity. For more security, never exceed 2/3 of the reservoir capacity.

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Low speed handpiece (HP-2)

1. Plug the unit
2. Screw the low speed handpiece onto the appropriated tubing with connector using gentle force with your hands.
3. The selection between low speed and high speed handpiece is done using a switch (HP1/HP2) on the control panel.
4. The low speed handpiece is operated by pneumatic variable foot control.
5. The low speed handpiece works only with air.

High speed handpiece (HP-1)

1. Plug the unit
2. Screw the high speed handpiece onto the appropriated tubing with connector using gentle force with your hands.
3. The selection between low speed and high speed handpiece is done using a switch (HP1/HP2) on the control panel.
4. The high speed handpiece is operated by pneumatic variable foot control.
5. The high speed handpiece can be used with or without water.
6. The water is activated or deactivated using (WATER ON/OFF) toggle on the control panel.
7. The high speed handpiece water flow is regulated by a knob (WATER ADJ.) on the control panel.

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CLEANING AND MAINTENANCE

Air/water Syringe

1. Clean and desinfect the syringe with a desinfectant that is used in dental offices.
2. The air/water tip as well as the syringe are autoclavable.

Oral evacuation system

1. Clean the oral evacuation system with disinfected.
2. The beak of the oral evacuation system can be put into the autoclave.
3. Unassemble the valve SAF#00321 each week by hand, clean and reassemble with lube (SAF#00221).
4. The same principle is used for the high volume (HVE).
5. HVE tip is autoclavable (optional with).

Water bottle

1. Purge the water bottle (see instructions page 6).
2. Clean with dish soap and rinse thoroughly.

Waste reservoir

1. Remove all tubing and empty the waste reservoir.
2. Clean with dish soap and rinse thoroughly.

Note: We recommend to clean and disinfect between each patient.

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Low and high speed handpiece

Follow the manufacturer instructions for disinfection and lubrication.

Tubing

Note : Between each patient, use maximum 4 ounces of non-foaming product per suction tubing, with the same quantity of water without exceeding 2/3 of the waste reservoir capacity, then empty the contents.

Filling the waste reservoir beyond 2/3 of its capacity could seriously damage the engine.

PURGE OF THE WATER BOTTLE

1. Depressurize the water bottle by unscrewing it gently.
2. Empty its contents in winter and summer.
3. Fill the bottle with water.
4. Screw on the water bottle.

STORAGE INSTRUCTIONS

1. Unplug the unit.
2. Place the air/water syringe in the storage space.
3. Empty and clean the waste reservoir following the cleaning instructions.
4. Empty and clean the water bottle following the cleaning instructions.
5. Place the rest of the equipment in the empty spaces.
6. Close the door.

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Net weight (Approx.) 31,1 lbs (14,06 kg)

TRANSPORT

Never carry or move the unit with the pressurized reservoir or any liquid.

The air and water reservoirs absolutely need to be emptied before transport.

WARRANTY

- ◆ This fine Safari Dental product is manufactured according to a rigid code of quality standards and, with a maximum standard of care, should give this unit years of satisfying service.
- ◆ Should the unit need repair or replacement parts (within or after the warranty period), please contact Safari Dental Inc. and we will direct you to the closest service center.
- ◆ Safari Dental Inc. units are warranted to be free from defects in material and workmanship for one (1) year from the date of purchase by the consumer. Any part of the equipment found to be defective under normal installation and use during the warranty period will be repaired or replaced by Safari Dental Inc. The equipment should be returned. Postage or freight prepaid to Safari Dental Inc. and must be accompanied by a proof of purchase.
- ◆ This warranty does not apply to equipment which has been subjected to abuse, accidents, alterations or defects due to unauthorized service and repair. Repair or replacement is the purchaser's sole remedy under this or any other warranty on the product whether express or implied

All equipment returned to Safari Dentaire Inc. must be insured against damage or loss.

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REPAIR RETURN PROCEDURE

We will assist you or direct you to return the product with a copy of the original purchase invoice to the nearest service center.

Carefully package the product to be returned to avoid any form of damage during transport. Return only the defective part(s). Keep all removable parts to avoid further problems during transport.

It is preferable to keep the cardboard box used for delivery, as they may be useful if you need to return the unit for repair.

If you need to return your unit, please include the following instructions:

- ◆ A brief description of the problem
- ◆ A contact name
- ◆ A contact telephone number, fax number or email address
- ◆ A complete address for the return of the unit
- ◆ If returned under warranty repair, please provide proof of purchase.

Upon receipt of the unit, our technicians will inspect the unit and you will be provided with an estimate of repair. The purchaser shall bear all shipping, packing and insurance costs. All repair work to be done after the warranty period, will be billed you for parts and labour only on a flat rate repair fee basis.

**Please do not hesitate to call if you have any questions.
Toll free: 1-800-567-0013 or 450-433-5754
Email: sales@safaridental.com**