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CAUTION: TO REDUCE THE RISK OF ELECTRIC SHOCK DO NOT REMOVE SAFETY COVERS. NO USER-SERVICEABLE PARTS INSIDE. REFER SERVICING TO QUALIFIED SERVICE PERSONNEL.

- To prevent fire or electric shock hazard, do not expose this product to rain or moisture.
- Do not block any ventilation openings. Install in accordance with the manufacturer's instructions.
- Improper installation may result in fire, explosion, electric shock, or injury. Incorrect use can cause serious hazards or injuries. You must follow all instructions.

- It is recommended that the unit be connect to a dedicated circuit;** That is, a single outlet circuit which powers only that unit and has no additional outlets or branch circuits. Do not overload wall outlets. Overloaded wall outlets, loose or damaged wall outlets, extension cords, frayed power cords, or damaged or cracked wire insulation are dangerous. Any of these conditions could result in electric shock or fire. Periodically examine the cord of your unit, and if its appearance indicates damage or deterioration, unplug it, discontinue use of the unit, and have the cord replaced with an exact replacement part by an authorized technician. Protect the power cord from physical or mechanical abuse, such as being twisted, kinked, pinched, closed in a door, or walked upon.

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IMPORTANT SAFETY INSTRUCTIONS

1. **Read these instructions** – All these safety and operating instructions should be read before the unit is operated.
2. **Keep these instructions** – The safety, operating and use instructions should be retained for reference.
3. **Heed all warnings** – All warning on the product and in the operating instructions should be adhered to.
4. **Follow all instructions** – All operating and use instructions should be followed.
5. Clean only with a clean cloth and products authorized by the manufacturer – Unplug this unit from the wall outlet before cleaning. Only use cleaning products used in dental and medical offices.
6. **Do not block any ventilation openings. Install in accordance with the manufacturer's instructions** – Slots and openings in the cabinet are provided for ventilation and to ensure reliable operation of the unit and to protect it from over-heating.
7. **Handle this unit with care – Do not hit or damage this unit during transportation.**
8. **Do not install near any heat sources such radiators, heat registers, stoves, or other apparatus that produce heat.**

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11. **There is an auxiliary socket to used to plug in other equipment. Total leakage current (3 AMP maximum), do not exceed 500 mA.**
12. **Do not defeat the safety purpose of the polarized or grounding-type plug.** A polarized plug has two straight blades with one wider than the other. A grounding type plug has two straight blades and a third u-shaped grounding prong. The wide blade or the third prong are provided for your safety. If the provided plug does not fit into your outlet, consult an electrician for replacement of the outlet.
13. **Only use attachments/accessories specified by the manufacturer.**
14. **Unplug this unit during lightning storms or when unused for long periods of time.**
15. **Refer all servicing to qualified service personnel.**
16. **See page 14 for periodic maintenance.**

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INSTALLATION AND INITIAL REGULATINGS

- 1. Place the unit on a flat surface and secure it with wheels with brake** - Carefully install the air / water syringe, the handpieces and suction valves on their support provided to this effect on the front. The handpiece has their dedicated holders because they are automatic.
- 2. Water reservoir preparation** - Unscrew the reservoirs and clean the water reservoirs (See instructions for cleaning). Fill the tank with clean water (preferably distilled water). Screw the reservoirs back on with your hand.
- 3. Fix the power cord to the unit and plug it correctly into a wall outlet.**
- 4. Turn on the unit by pressing the On/Off button on the front of the unit.**
- 5. Wait until the pressure builds in the unit before use.**

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INSTRUCTIONS UTILISATION

Syringe air/water

1. Check jet of air, water, and air / water spray when the compressor is activated.

Note : The air compressor feeding the syringe air / water will automatically run and stop when the air reservoir will be full.

Handpieces connectors (2)

1. Screw a high-speed handpiece onto the appropriated tubing with connector using gentle force with your hands.
2. The selection between handpieces is automatic.
3. The handpieces are operated by pneumatic variable foot control.
4. The handpieces can be used with or without water.
5. The water is activated or deactivated using (WATER ON/OFF) toggle on the control panel.
6. The handpiece water flow is regulated by a knob (WATER ADJ.) on the control panel.

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DISINFECTION GUIDELINES

Safari Dental makes no representation as to the disinfection efficacy of any disinfectant product. Questions regarding product efficacy should be directed to the specific manufacturer of that product. We make no warranty expressed or implied regarding any damage resulting from the use of any disinfectant. Damage and/or discoloration of any surface is not covered by our warranty.

There are many products and techniques for infection control that are presently available on the market.

The **BARRIER TECHNIQUE METHOD** has proved to be effective in the preservation of the surface finish and appearance of your new equipment. Disposable barrier products should be used and changed between each patient. Consult your full-service dental dealer for information regarding disposable barrier products.

The **CHEMICAL DISINFECTION METHOD** is an effective means of disinfection, but care is needed in the mixing and use of the products. To minimize the accumulation of chemical residues on your new equipment, you must wash your equipment daily with a solution of warm water and mild soap. When mixing any cleaning or disinfecting chemical product, care should be taken to follow the manufacturer's instructions regarding mixing and use. If recommended ratios of dilution are not followed, non-damaging chemical may become corrosive and damage surface finishes. Consult your full-service dental dealer for information regarding chemical disinfecting and cleaning products.

Non-Acceptable disinfectants and cleaners include Household bleach @ 5.25%, Ethyl alcohol, alcohol, alcohol-based phenols, household cleaners and strong phenols or alcohol-phenol combinations. The use of iodine-based products has been known to stain painted and upholstered surfaces.

When **FLUSHING UNIT LINES AND WATER BOTTLE SYSTEMS**, care must be taken. If a powdered flushing product that requires mixing with water is used, the powder must be thoroughly dissolved in the water. Improper mixing will cause solution particulates. These particulates may cause residual debris in the water lines which will cause the dental system to not work properly.

DAMAGE OR DISCOLORATION OF SURFACE FINISHES IS NOT COVERED UNDER THE SAFARI DENTAL WARRANTY.

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CLEANING AND MAINTENANCE PROCEDURE

Expedition unit is made for easy cleaning and disinfection procedures. All contact surfaces that are not barrier-protected, can be clean and disinfect^{by} using registered hospital disinfectant with low- (i.e., HIV and HBV label claims) to intermediate-level (i.e., tuberculocidal claim) after each patient.

Self-Contained water system

The self-contained water system ensures a dependable source of pure water. The use of Heavy-Duty Bottle assures safety and minimizes the chance of leakage. The bottle is made of High-Density Polyethylene (HDPE) and is compatible with coolant liquids used in dentistry.

To get cleaned water, we recommend changing water after each patient, purge the system (see instructions) and refill with distilled water. Clean with dish soap and rinse thoroughly.

For waterline treatment, we recommend the use of lines. Get it from your dental Dealer. It will even improve your dental unit performance

Use distilled water. When not available, use water that meets standards for drinking water (i.e., <500 CFU/mL of heterotrophic water bacteria)

Discharge water and air for a minimum of 20–30 seconds after each patient, from any device connected to the dental water system that enters the patient's mouth (e.g., handpieces, ultrasonic scalers, and air/water syringes)

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Instrumentation: Syringe and Tubing

Air Syringe: smooth contours for quick, thorough cleaning. Use standard autoclave tips or disposable tips. Clean and disinfect the syringe with registered hospital disinfectant. The air/water tip as well as the syringe are autoclavable.

We use only straight asepsis polyurethane tubing made by FreelinWade. These tubing's are smooth (inside and outside) and inhibit the growth of bacteria on their surfaces. They are easy to clean and minimizes need of aggressive corrosive chemicals. All contact surfaces that are not barrier-protected, can be clean and disinfect^{by} using registered hospital disinfectant with low- to intermediate-level after each patient.

Barriers

Use surface barriers to protect clinical contact surfaces, particularly those that are difficult to clean (e.g., switches....) and change surface barriers between patients. For more protection, we also recommend the use of barriers for handpiece tubing, 3-way syringe and oral evacuation tubing.

Clean air

Expedition unit is equipped with an oil-less clean air compressor and use a unique Stainless Steel air reservoir. No rust (Unique to Safari)

To get 100 % dry air, purge air reservoir after each patient. To do so, turn off the compressor switch. Push the button "DRAIN" in to release all air in unit. Repeat this procedure if sing of humidity still appears.

Handpiece

Follow the manufacture instructions for disinfection and lubrication.

Motors

motors are sealed and require no lubrication. If the motor does not start, do not try to open, or repair. All repairs incorrectly entail risk of electric shocks. It is recommended to return to the distributor or directly to SAFARI DENTAL INC.

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Instruction Scaler

1. Sterilization and maintenance

1.1 Sterilization of the detachable handpiece

1.1.1 Autoclave at high temperature/pressure :

- a) 121 °C/lbar (0.1MPa)
- b) 135 °C/2.2bar (0.22MPa)

1.1.2 Remove the handpiece and clean the tip and endo chuck after the procedure.

1.1.3 Wrap the handpiece with sterile gauze or a sterile bag before sterilization.

1.1.4 Reuse the handpiece after it has cooled naturally in case of hand burns.

1.1.5 Notice:

a) Clean the cleaning fluid from the handpiece with compressed air before sterilization.

b) Make sure that the scaling tip has been unscrewed from the handpiece and that there are no mistakes.

c) Please note if the outside of the handpiece is damaged during processing and sterilization. Do not apply any protective oil to the surface of the handpiece.

d) There are two sealed O-rings at the end of the handpiece. Please lubricate them frequently with dental lubricant, as sterilization and repeated insertion and removal reduce their life. Replace them with new rings when they are damaged or excessively worn.

e) The following sterilization methods are prohibited:

- 1- Put the handpiece in a boiling liquid.
- 2- Put the handpiece in a disinfectant such as iodine, alcohol and glutaraldehyde.

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3- Place the handpiece in the oven or microwave for cooking.

1.2 Sterilization of the scaling tips

All decalcifying tips can be sterilized with an alcohol-soaked cotton ball or a disinfected cloth. They can also be sterilized with an ultrasonic cleaner.

1.3 Sterilization of the torque wrench

1.3.1 The torque wrench can be sterilized with a neutral, non-corrosive disinfectant for cleaning and sterilization, or it can be sterilized at high temperature and under pressure.

1.3.2 The following means of sterilization for the torque wrench are prohibited:

- a) Boiled in liquor.
- b) Soak in iodine, alcohol and glutaraldehyde.
- c) Roasted in oven or microwave

Notice: We are not responsible for any damage caused by the above item.

1.4 Cleaning the Torque Wrench Tips

The chipping tip of the torque wrench can be cleaned ultrasonically.

1.5 Sterilization and cleaning of the LED lamp and light pipe

Clean the LED lamp and light pipe with purified water and sterilize them at high temperature and high pressure after each use.

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Fault	Possible causes	Solutions
The scaling tip doesn't vibrate when stepping on the foot switch.	Handpiece and the connector of cable connect irrelevantly	Pull out handpiece and insert it again.
	Scaling tip is loose	Screw it on tightly with torque wrench.
	There is some water between the handpiece and the connector of cable	Dry the connect point.
	There is something wrong with detachable	Send it to our company to repair
The scaling tip vibrates, but there is no water flowing	Water supply of dental unit is off	Check the water supply of the dental unit.
	There is no water coming out from the cable	Clean the water pipe of the cable with multi-fuction syringes
	There is no water coming out from the handpiece	Clean the water pipe of the handpiece with multifunction syringes
The handpiece generates heat	The amount of spouling water is too little	Turn the water control switch to a higher grade

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Fault	Possible causes	Solutions
the amount of spouting water is too little	The water pipe of dental unit is jammed	Clean the water pipe
	the water pipe cable is jammed	Clean the water pipe of the cable with multi-function syringe
	The water pipe of handpiece is jammed	Clean the water pipe of the handpiece with multi-function syringe
	The water pressure is not high enough	Increase water pressure
The vibration of the tip becomes weak.	The tip hasn't been screwed tightly	Screw down the scaling tip
	The tip vibrates loose	Screw down the scaling tip
	The tip is damaged	Change a new one
There is water seeping from the coupling between the handpiece and cable	The waterproof "O" ring is damaged.	Change a new "O" ring
LED light don't work	Poor contact	Contact tightly
	Something wrong with LED light	Change a new one
	LED lamp installed backwards	Please install the "+" of the LED lamp to the "+" of the handpiece
The potentiometer is failure.	The potentiometer is damaged	Change a new one

If the problem still can't be solved, please contact with local dealer or manufacturer

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2. Preparation

Notice when using the equipment

- 2.1** Keep the scaler clean before and after use.
- 2.2** The handpiece, scaling tip, torque wrench, endo wrench and endochuck must be sterilized before each treatment.
- 2.3** Do not screw or unscrew the decalcifying tip while walking on the foot pedal.
- 2.4** The scaling tip and endochuck must be secured and a fine stream of drops must come out of the tip when used.
- 2.5** Change a new one when the nozzle is damaged or excessively wormed. Do not twist or rub the nozzle.
- 2.6** During operation of the descaler, the heat of the descaling nozzle will increase if there is no water flowing, please maintain a steady water flow.
- 2.7** Do not use impure water source and make sure not to use normal brine instead of pure water source.
- 2.8** Ensure that the handpiece connector and cable plug are dry before installing the handpiece.
- 2.9** Do not pull the cable by force in case the handpiece falls off the cable.
- 2.10** The internal screw thread of the descaling tips produced by some other manufacturers, may be rough, rusty, and collapsed. This will irreparably damage the outer screw of the handpiece. Please use "DTE" brand descaling tips.
- 2.11** Before connecting the built-in ultrasonic scaler without transformer to the power supply, please check that the output voltage is 24V-, in case of connection to the wrong power supply, the unit may be damaged.
- 2.12** Dental unit manufacturers or end user are not allowed to disconnect the built-in ultrasonic scaler in case it affects the function of the scaler. If you have a special request, please contact us.

Warning: No modification of this equipment is allowed.

Warning: If this equipment is modified, proper inspection and testing must be performed to ensure continued safe use of the equipment.

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STORAGE INSTRUCTIONS

1. Empty and clean the waste container and tubing following the cleaning instructions.
2. Press the "DRAIN" button on the left side of the unit until there was no moisture below the unit
3. Empty and clean the water tank following the cleaning instructions and put it back in place, empty.
4. Empty the remaining water inside the tubing
 - 3-way syringe tubing
 - Handpiece tubing 1-2
 - Scaler
5. Disconnect and clean your instruments
(handpieces, syringe tip ...)
6. Unplug the power cord and store it.
7. Place the instruments on the supports provided for that purpose.

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Operating humidity: 5 % - 75 %

WARRANTY

- This fine Safari Dental product is manufactured according to a rigid code of quality standards and, with a maximum standard of care, should give this unit years of satisfying service.
- Should the unit need repair or replacement parts (within or after the warranty period), please contact Safari Dental Inc. and we will direct you to the closest service centre.
- Safari Dental Inc. units are warranted to be free from defects in material and workmanship for one (1) year from the date of purchase by the consumer. Any part of the equipment found to be defective under normal installation and use during the warranty period will be repaired or replaced by Safari Dental Inc. The equipment should be returned. Postage or freight prepaid to Safari Dental Inc. and must be accompanied by a proof of purchase.
- This warranty does not apply to equipment which has been subjected to abuse, accidents, alterations, or defects due to unauthorized service and repair. Repair or replacement is the purchaser's sole remedy under this or any other warranty on the product whether express or implied.
- Any goods returned to Safari Dental Inc. should be insured against damage or loss as we cannot be held responsible for any such damage or loss by the purchaser, carriers, or agents.**

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Courriel: sales@safaridental.com

REPAIR RETURN PROCEDURE

We will assist you or direct you to return the product with a copy of the original purchase invoice to the nearest service centre.

Pack your product carefully to avoid damage in shipping. Only send the defective part. Keep all removable parts to avoid further damage during the transport.

Note : It is better to retain the delivery box, since they could be useful if you had to return the unit for service.

If you need to return your unit, please include the following instructions:

- A brief description of the problem
- A contact names
- A contact telephone number, fax number or email address

- A complete address for the return of the unit
- If returned under warranty repair, please provide proof of purchase.

Upon receipt of the unit, our technicians will inspect the unit and you will be provided with an estimate of repair. The purchaser shall bear all shipping, packing and insurance costs. All repair work to be done after the warranty period, will be billed you for parts and labour only on a flat rate repair fee basis.

Please do not hesitate to call if you have any questions.

Toll free: 1-800-567-0013 or 450-433-5754

Email: sales@safaridental.com