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## **Checklist for high and low volume suction**

- You must use the least foaming solution possible and ideally the non-foaming solution. The recommended quantity is half a "dixie cup" per tubing and preferably between each patient, or according to your protocol (beginning and end of the day).
- When the suction motor is activated, it should start instantly. If, however, the engine hesitates to start, the filter/muffler could be the cause and in the worst case the engine will not start at all and could have a burning smell, because it is impossible to know if the motor is on or off this means that the filter/muffler is completely blocked and requires a filter/muffler change. If you don't have a filter/muffler available, you can temporarily help yourself by drilling a hole the size of the air/water gun (you can use a curette to drill the hole) through the filter/muffler into the white part.
- If you use the two valves at the same time, put the two levers at half and even a little more for the high volume to allow suction to the two tubes, but it is strongly recommended to use one at a time for a maximum efficiency.
- To maximize suction power, close the lever halfway.
- It is suggested to empty the waste jar when it is half full.
- Do not close both suction valves when the suction motor is running, keep one valve open.
- At the end of the day, throw away the contents of the waste jar and rinse with clean water, in addition, suck only the solution.
- Never transport the unit with liquid either water or waste.